

ID of *E.coli* O157 made simple!

new from Hardy Diagnostics...




Compact size conserves valuable refrigerator space.




EcoliPRO™ O157 uses latex agglutination to offer definitive identification of *E. coli* O157 from a single isolated colony. Results are obtained in two minutes. This kit is designed to test all O157 serotypes, which produce verotoxins including, but not limited to, the O157:H7 serotype.


- ✓ **RAPID** Accurate results in two minutes at room temperature.
- ✓ **SENSITIVE** As little as one colony required.
- ✓ **SPECIFIC** Eliminates the possibility of cross reactions with *Escherichia hermannii* and other sorbitol negative *Escherichia* species.
- ✓ **CONVENIENT** All reagents are ready-to-use and remain in one space-saving workstation.
- ✓ **COMPLETE** Positive and negative controls are included, along with mixers and cards.
- ✓ **DURABLE** Two year shelf-life from the date of manufacture.

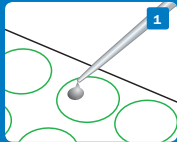

1 Add one drop of saline to a test circle.

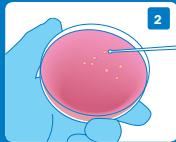


2 Pick 1 to 4 colonies from your petri plate.

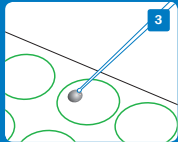

3 Mix bacteria with the saline in test circle.



4 Add one drop of latex reagent to test circle.

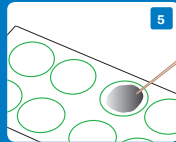

5 Mix with stick. Rock and read within 2 minutes.

EcoliPRO™ O157 Kit

50 Test Kit.....PL070HD

HARDY
DIAGNOSTICS

A culture of service . . .

EcoliPRO™ 0157 Kit



1430 West McCoy Lane
Santa Maria, CA 93455

I would like more information on this product.

Please fill in your information below and fax the completed card to us at 805.346.2760.

Name _____

Title _____

Company _____

Address _____

City _____

State/Zip _____

Phone _____

Email _____

For more information on EcoliPRO™ 0157 Kit or any other Hardy Diagnostics products:

Fax the completed card to 805.346.2760.

Call toll free at 800.266.2222 ext. 5696

Email us at Sales@HardyDiagnostics.com

Visit us online at www.HardyDiagnostics.com