

# Simplify and Streamline

## your fungal slide cultures



- Save money and reduce frustration by eliminating preparation time.
- Prevents desiccation with a built-in humidity chamber; can be incubated for long periods of time.
- Device can fit on the microscope stage for easy observation without disturbing the delicate fungal structures.



# MycoVue™

**HARDY**  
DIAGNOSTICS  
*A Culture of Service™*



1

Remove cover and tab.



2

Inoculate with specimen.



3

Place sterile cover slip on medium.




4

Add water to humidifier.



5

Replace lid and incubate.

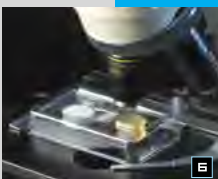

6

Read microscopically.

MycoVue™ is a ready-to-use slide culture system for fungi. It features a built-in humidity chamber and is designed to fit on your microscope stage for easy observation.

### MycoVue™ Potato Flake Agar

A general purpose culture medium for all fungi. Induces rapid sporulation. 5/pk .....MV1

# Mycovue™

**I would like more information on these products.**  
Please fill in your information below, snap a photo of this postcard and send it to [Sales@HardyDiagnostics.com](mailto:Sales@HardyDiagnostics.com)

Name \_\_\_\_\_

Title \_\_\_\_\_

Company \_\_\_\_\_

Address \_\_\_\_\_

City \_\_\_\_\_

State/Zip \_\_\_\_\_

Phone \_\_\_\_\_

Email \_\_\_\_\_

**For more information and ordering MycoVue™ and other Hardy products visit our website at [www.HardyDiagnostics.com/Mycovue.html](http://www.HardyDiagnostics.com/Mycovue.html) or scan our QR code.**

Call toll free at 800.266.2222 ext. 5696

Email us at [Sales@HardyDiagnostics.com](mailto:Sales@HardyDiagnostics.com)



**HARDY**  
**DIAGNOSTICS**  
A Culture of Service™

1430 West McCoy Lane  
Santa Maria, CA 93455